



PHILIPS

Zoom!

Teeth whitening



Love your
smile

© 2018 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice. Trademarks are the property of Koninklijke Philips N.V. (Royal Philips) or their respective owners.

www.philips.co.uk/zoom

Philips Zoom! Take-Home Whitening



Getting started with
your Philips Zoom! at
home teeth whitening



What you will find in your kit:

- Custom-fitted whitening trays
- Whitening gel syringes
- Reusable mixing tips
- Tray storage case
- Shade guide
- Patient Instructions

Step 1:

Assemble your syringe

In an anticlockwise motion, unscrew the clear plastic cap from the syringe and remove.

Take one of the small mixing nozzles, which you will find within your whitening tube pack. Push the mixing nozzle on to the end of the syringe and twist clockwise to lock it in. Finally, remove the rubber tip.

Step 2:

Put the gel in the whitening tray

Place a small amount of gel (rice grain sized) along the front edge of each tooth compartment of your whitening tray. Do not overfill the tray.



Step 3:

Prep your mouth

Floss and brush your teeth before starting your whitening treatment. Moments before you put the tray in your mouth, dry your teeth thoroughly.

Step 4:

Wear your tray

Place the tray with the gel in it in your mouth, so the tray fits comfortably around your teeth. Bubbling within the tray is normal.

If there is excess gel, wipe this away with a cotton wool bud or tissue.

Leave your whitening tray in place for the amount of time advised by your dentist.

Note: Do not eat, drink or smoke while wearing your whitening trays.

Step 5:

After the advised wear time is up

Rinse and brush your teeth to remove excess gel.

Step 6:

Storing your whitening kit

Once you've finished placing the whitening gel in your trays, remove the mixing nozzle from the syringe and replace it with the clear plastic cap.

Rinse your whitening trays with cold water, using a toothbrush to remove any residual gel if needed. Place your trays in the storage case.

Return your whitening trays and syringes to the bag and store ready for your next use.



Recommended Wear Times:

Philips Zoom! NiteWhite

Formulation: 10% or 16% Carbamide Peroxide

Wear time: Overnight or 4-6 hours

Results: Optimal results within 1-2 weeks

Philips Zoom! DayWhite

Formulation: 6% Hydrogen Peroxide

Wear time: 60 minutes, once daily

Results: Optimal results within 1-2 weeks